

## **Bite Record for Superior Repositioning Splints**

- 1) Select Dobson from drop down list on iTero scanner
- 2) Scan both arches with scan extending to at least the gingival margins on the labial surfaces in both arches and lingual surfaces on the lower arch and 5 mm of palatal coverage in the upper.
- 3) Take Kois face bow to orient the maxilla on the virtual articulator
- 4) Scan Kois plate and spray with white powder (anit-fungal foot spray works well)
- 5) Wax bite made from four thicknesses of Delar wax trimmed to 1 inch long
- 6) Place wax in 140\* water bath until dead soft
- 7) Recline patient to supine position with head tipped back
- 8) Place softened Delar wax on upper anterior teeth
- 9) Guide the mandible closed into the Delar wax until to posterior teeth are about 2 mm apart
- 10) Remove the wax and place in ice water to harden
- 11) Replace the wax and have the patient close again into the wax that now serves a template
- 12) With the chilled wax in the mouth and patient slightly squeezing, place the scanner wand in the buccal corridors and capture the interocclusal space on each side when prompted by the scanner to "bite."